

# **Chondroprotec®**

Polysulfated Glycosaminoglycan, Sterile Solution





### **Indications**

Chondroprotec is a sterile 10 mL solution containing 1000 mg of Polysulfated Glycosaminoglycan (100 mg/mL).

For use on dogs, cats and horses.

- Partial and full thickness wounds
- Secreting and bleeding dermal lesions
- Trauma injuries
- 1st and 2nd degree burns
- Pressure ulcers (Stages I-IV)
- Surgical incisions
- Donor sites
- Venous stasis ulcers

# **Presentation** R

08820 1000 mg/ 10 mL vial 3 vials/ box

## **Dosage and Administration**

Irrigate wound with normal saline solution.

Either pour contents of vial or dispense from syringe to withdraw solution and cover wound site.

Cover with a non-adherent dressing such as PUF (polyurethane film) or Telfa®. Note: Subsequent applications of Chondroprotec can be injected through the dressing as to not disturb the wound bed.

Reapply Chondroprotec daily and redress wound site every 5-7 days or as needed.

#### **Features**

1000 mg of PSGAG in a 10 mL vial (100 mg/mL)

Derived from bovine trachea

Preservative free

Patented technology

#### Caution

Not for food animal use.

Not for human use

No known side effects; not intended for use as a long term or permanent dressing.

Federal law restricts this device to sale by or on the order of a licensed veterinarian.

## **Storage**

Store at a temperature between 15°-30°C (59°-86°F).

Protect from freezing.

Sterile when unopened.

Discard unused portion.

