

INNOVACYN, INC.

3546 N. RIVERSIDE AVE., RIALTO, CA, 92377

Toll-Free: 866-318-3116

Fax: 909-428-1947

Website: www.innovacyn.com

Website: www.vetericyn.com

Email: customerservice@innovacyn.com

Every effort has been made to ensure the accuracy of the information published. However, it remains the responsibility of the readers to familiarize themselves with the product information contained on the USA product label or package insert.

VETERICYN PLUS® ANTIMICROBIAL HYDROGEL

Innovacyn

VETERINARIAN RECOMMENDED

relieve itch, pain, and protect skin to help healing of wounds | cuts | sores | skin irritations

- Gels on contact & provides protection
- Non-toxic, safe if licked or ingested
- Relieves symptoms of skin allergies
- No alcohol, steroids, or antibiotics

Indications: Vetericyn Plus® Antimicrobial Hydrogel is intended for the management of wounds including skin abrasions, lacerations, irritations, cuts, burns and intact skin; providing a proper moisture balance throughout the phases of wound healing. Relieves itch and pain associated with skin irritations. Safe for all animal species.

Storage: Store at room temperature away from direct sunlight and heat. Do not allow to freeze. Do not use after expiration date. After use, rinse nozzle with warm water to reduce chance of hydrogel drying at the tip. Replace cap (3 fl oz and 8 fl oz). Turn sprayer nozzle to Stop Position (16 fl oz).

Stability: See Expiration Date

Disposal: No special disposal requirements. Vetericyn Plus is environmentally friendly.

Directions for Use: To prime for initial application, remove cap, depress and hold sprayer, shake bottle, release sprayer. Move excess hair from wound area. Clip if necessary. Saturate the affected area. If dressing is required, saturate dressing with Vetericyn Plus® Wound & Skin Care or Antimicrobial Hydrogel at each dressing change. Repeat 3-4 times per day or as necessary. Faster healing is achieved when wounds are cleaned and maintained moist. No rinsing necessary. Safe for use around mouth, nose, ears, and eyes. For best results, first clean affected area with Vetericyn Plus Wound & Skin Care to ensure deep dermal cleansing and proper wound bed preparation.

Use Precautions: Safe for all animal skin types at all life stages. For external use only, not for injection. Always consult with a veterinarian for systemic conditions. Discontinue use if irritation develops and contact your veterinarian. For animal use only, not for human use. Keep out of reach of children. Safe if licked by animal. Will not stain and contains no harsh chemicals or dyes.

Ingredients: Active: Hypochlorous Acid (0.010%) **Inactive:** Electrolyzed Water, Sodium Chloride, Sodium Hypochlorite, Lithium Magnesium Sodium Silicate, Sodium Bicarbonate, Phosphates, Sodium Sulfate

Innovacyn products are manufactured under strict quality guidelines of ISO 13485 and FDA QSR. Vetericyn Plus Hydrogel formula contains Hypochlorous Acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms in the Hydrogel.

Made in the USA

Manufactured & Bottled by Innovacyn, Inc., 3546 N. Riverside Ave, Rialto, CA 92377

vetericyn.com

866.318.3116

Tell Us Your Story!

	Part Number:
3 fl oz (89 ml)	1047; 10047 v1.3
8 fl oz (237 ml)	1042; 90030 v5.3
16 fl oz (473 ml)	1040; 90007 v5.3

CPN: 1433005.9

By using this content, you agree to the [Terms of Service](#) and [Privacy Policy](#).

Copyright 2020 - Animalytix LLC